### NOTICE FOR USE OF THE ESTELITE Σ QUICK SHADE GUIDE

- Keep the ESTELITE Σ QUICK Shade Guide away from direct sunlight to prevent a change in color.
- Avoid contact between the ESTELITE Σ QUICK Shade Guide and bonding agents, etc., because the surface of the ESTELITE Σ QUICK Shade Guide will dissolve if it comes into contact with methacrylic monomers or solvents.

The ESTELITE Σ QUICK Shade Guide is composed of the following materials:

Shade piece: Polystyrene

Base plate: ABS (Acrylonitrile-Butadiene-Styrene) resin

- Refer to the following table when cleaning the ESTELITE Σ QUICK Shade Guide.

	Cleaning method
Correct	<ul style="list-style-type: none"><li>wash with soap and water</li> <li>ultrasonic cleaning while immersed in water</li> <li>immerse in a 3% glutaraldehyde solution</li> <li>wipe with a cotton swab soaked in an alcohol (80%) solution</li></ul>
Incorrect	<ul style="list-style-type: none"><li>autoclave<sup>1)</sup></li> <li>dry heat sterilization<sup>1)</sup></li> <li>immerse in an alcohol (80%) solution<sup>2)</sup></li> <li>immerse in a sodium hypochlorite solution<sup>3)</sup></li> <li>immerse in a hydrogen peroxide solution<sup>3)</sup></li></ul>

- Shade piece and base plate will melt.
- Base plate will whiten and crack. Shade indications and the logo will fade out.
- Shade piece might be decolorized.

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