TOKUPOST.18

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INSTRUCTIONS FOR USE OF ENDOCANALAR TRANSLUCENT QUARTZ FIBER POSTS

INSTRUCTIONS FOR USE

1. DESCRIPTION

TOKUPOST Iperquartz is a highly translucent quartz fiber radio-opaque post, with variable conicity. It is available in two lengths and shapes: 18 with 18 mm length - 15R with 15 mm length and coronal retentions.

2. INDICATIONS FOR USE

- The use of these devices is intended only for odontologists and odontostomatologists;
- In case of insufficient residual dental substance (< 4 mm), TOKUPOST Iperquartz is suitable to sustain the restoration of coronal walls.

3. CONTRAINDICATIONS

In case of insufficient residual dentine, ("ferule" effect), less than 2 mm, the use of posts is not recommended.

4. WARNINGS

TOKUPOST Iperquartz posts are disposable medical devices; it is absolutely forbidden to reuse an already used post. This would entail not only serious risks of bacterial contamination and infections transmission, but also of the collapse of the post already subject to previous physical-mechanical stress.

5. PRECAUTIONS

It is recommended to clean the post with alcohol before placing it into the canal, in order to "degrease" it if it has been decontaminated. The TOKUPOST Iperquartz post is silanized. If necessary cut the post outside the oral cavity (TOKUPOST 18)

6. UNWANTED REACTIONS

Based on literature available to date, no unwanted reactions are known.

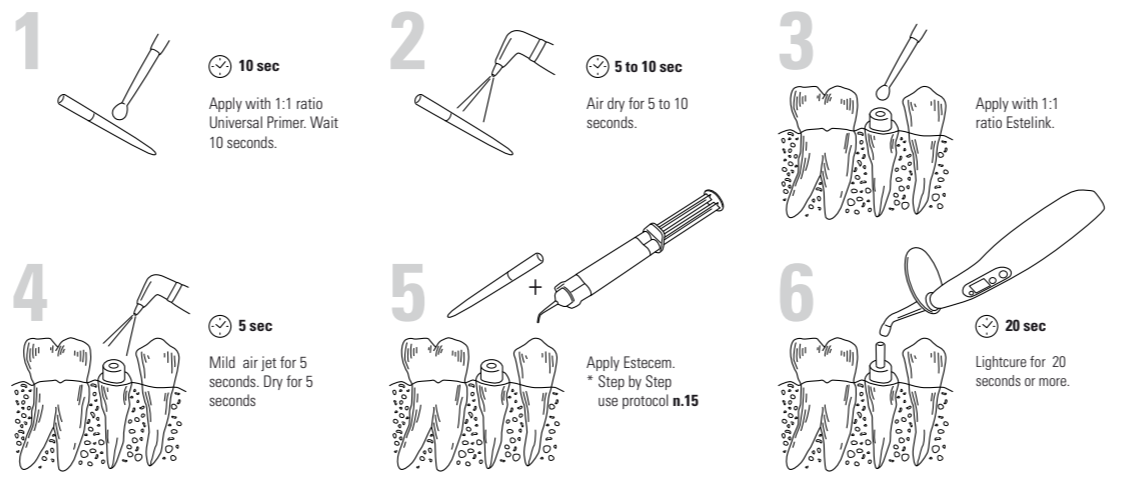
7. DISPOSAL OF THE DEVICE

The devices to be disposed must be treated as per the applicable regulations (law decree 152/2006 and following modifications and additions), and as per the waste treatment regulation of local entities. Lacking different specific instructions, dispose of the device as special hospital waste.

8. SAFETY AND LIABILITY

- Tokuyama Dental accepts no liability or claim for damages due to:
- use of instruments not belonging to the system and which might compromise its operation;
- non compliance with the instructions for use.

! The user is liable for using materials for uses different from the ones specified in the instructions for use.

EXPLANATION OF SYMBOLS ON THE LABEL

CE mark	Warning (generic risk)	Reference code
Manufacturer	Disposable	Production lot

Tokuyama Dental Italy s.r.l.
Via Chizzalunga, 1
36066 Sandrigo (VI) Italy
info@tokuyama.it - www.tokuyama.it

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