Resin-based Dental Restorative Material ESTELITE UNIVERSAL FLOW [SYRINGE] REF PE01 ESTELITE UNIVERSAL FLOW [PLT]

Last updated 2021-04-20 Rev.D

ENGLISH

Carefully read all information, precautions, notes and materials before using ESTELITE UNIVERSAL FLOW.

PRODUCT DESCRIPTION AND GENERAL INFORMATION

- 1) ESTELITE UNIVERSAL FLOW is a low viscosity, light-cured, radiopaque composite resin for use in anterior and posterior restorations. Direct placement of ESTELITE UNIVERSAL FLOW into a prepared cavity allows for easy handling.
- 2) ESTELITE UNIVERSAL FLOW incorporates Radical-Amplified Photopolymerization initiator technology (R.A.P. technology). R.A.P. technology facilitates a shortened light curing time and ample working time for the practitioner to determine the relationship between curing time and increment depth.
- [INDICATIONS FOR FILLING AND CURING]**
- 3) ESTELITE UNIVERSAL FLOW comes in three viscosities, High, Medium and Super Low. Each type contains a glass-filler and other shades are indicated in below table. The composite filler contains reduced polymerization shrink stress due to its round shape. The spherical filler contained in ESTELITE UNIVERSAL FLOW (mean particle size: 200 nm, particle size range: 100 to 300 nm) facilitates excellent gloss retention and wear resistance.

Type	Filter Loading
High	69% by weight (55% by volume)
Medium	71% by weight (57% by volume)
Super Low	70% by weight (56% by volume)

4) ESTELITE UNIVERSAL FLOW contains Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA), Bisphenol A poly(ethylene methacrylate) (Bis-MEEP), Triethylene glycol dimethacrylate (TEGDMA), 1,6-hexamethyl-di(2-hydroxy)benzylamine/methyl hexane (UDMA), Mequinol, Dibutyl phthalate and UV absorber.

5) ESTELITE UNIVERSAL FLOW is available in either SYRINGE (L-SYRINGE) or SYRINGE (Medium only) or Pre Loaded Tip (PLT (Medium only)).

[SHADE]
Available shade in each type is as below.

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- Opalescent shades (OPA2, OPA3, OPA4) have adequate opacity designed for blocking out the dark shade that comes through the oral cavity (Class III and IV restoration). The opalescent shades also are designed for maximum light stain or reconstructing a highly opaque tooth. Please check with your local distributor for available shades.

- [INDICATIONS]**
- Direct anterior and posterior restorations
 - Cavity base or liner
 - Blocking out cavity undercuts before fabricating indirect restorations
 - Repairs of porcelain veneers

[CONTRAINDICATIONS]
ESTELITE UNIVERSAL FLOW contains methacrylic monomers and UV absorber. DO NOT USE ESTELITE UNIVERSAL FLOW for patients allergic to or hypersensitive to methacrylic and related monomers, to UV absorber or to any of the other ingredients.

[PRECAUTIONS]

- 1) DO NOT USE ESTELITE UNIVERSAL FLOW for any purpose other than those listed in these instructions. Use ESTELITE UNIVERSAL FLOW only as directed herein.
- 2) ESTELITE UNIVERSAL FLOW is composed of materials that are safe and have been licensed dental care professionals only.
- 3) DO NOT USE ESTELITE UNIVERSAL FLOW if the safety seals are broken or appear to have been tampered with.
- 4) ESTELITE UNIVERSAL FLOW shows an allergic reaction or hypersensitivity, discontinue its use immediately.
- 5) Use examination gloves (plastic, vinyl or latex) at all times when handling ESTELITE UNIVERSAL FLOW to avoid the possibility of allergic reactions from methacrylic monomers or UV absorber. Note: Certain monomers may cause allergic reactions even though examination gloves are used. If ESTELITE UNIVERSAL FLOW comes in contact with the skin, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- 6) Avoid use of ESTELITE UNIVERSAL FLOW with eyes, mucous membrane, skin and clothing.
- 7) If ESTELITE UNIVERSAL FLOW comes in contact with the skin or clothing, immediately saturate the area with an alcohol soaked cotton swab or gauze.
- 8) Instruct the patient to rinse his mouth immediately after treatment.
- 9) To prevent the inhalation of dust, ESTELITE UNIVERSAL FLOW should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- 10) To avoid the unintentional ingestion of ESTELITE UNIVERSAL FLOW, do not leave it unsupervised within the reach of children.
- 11) When using a light-curing unit, protective eye shields, glasses or goggles should be worn at all times.
- 12) This product is designed to be used at room temperature (18 - 30°C / 62 - 84°F). Allow product to reach room temperature prior to use. Cold material may be difficult to extrude.

[INDICATIONS FOR FILLING AND CURING]

ESTELITE UNIVERSAL FLOW is designed to be cured by either a halogen, LED or Plasma Arc light with a curing time of 10 seconds. Be sure to use the PLT (Pre Loaded Tip) to reach the time needed for complete hardening of ESTELITE UNIVERSAL FLOW with your light-curing unit before performing the bonding procedure. The following table summarizes the relationship between curing time and increment depth.

Relationship between curing time and increment depth:

Type	Intensity (mW/cm²)	Curing time (seconds)	Increment depth (mm) ¹⁾		
			CE, BW, A1, A2, A3, A3.5, A4, B3	A5	OPOA2, OPOA3, OPOA4
Light type	600	10	1.8	1.6	1.3
		20	2.3	2.0	1.6
Halogen	800	10	2.0	1.7	1.5
		20	2.4	2.1	1.7
LED	1200	5	1.8	1.6	1.3
		10	2.2	2.0	1.6
Plasma Arc	600	10	1.7	1.5	1.2
		20	2.1	1.8	1.4
Plasma Arc	800	10	1.9	1.7	1.3
		20	2.2	2.0	1.6
Plasma Arc	1200	10	2.0	1.8	1.5
		20	2.3	2.1	1.7
Plasma Arc	2300	3	1.7	1.5	1.2
		10	2.2	2.0	1.6
Plasma Arc	1200	3	1.7	1.5	1.1

1) Increment depth was determined on the basis of test results performed in accordance with "depth of cure" ISO 4049.

[SPECIFIC NOTES FOR THE USE OF PLT]

1) PLT is not designed for single point use only, to avoid cross infection. Do not re-cap and/or re-use the PLT if your material has been dispensed for that patient.

2) Dispensers are not provided for ESTELITE UNIVERSAL FLOW. Use a dispenser that fits the PLT of ESTELITE UNIVERSAL FLOW. For proper use and proper disinfection, see manufacturer's instructions.

3) Use low controlled pressure to prevent any continued extrusion of material following pressure release.

[STORAGE]

1) Store ESTELITE UNIVERSAL FLOW at temperatures between 0 - 25°C (32 - 77°F).

2) AVOID direct exposure to light and heat.

3) DO NOT USE ESTELITE UNIVERSAL FLOW after the indicated date of expiration on the syringe or PLT package.

To avoid disposal of excess ESTELITE UNIVERSAL FLOW, extrude unused portion from SYRINGE or PLT and light-cure before disposal.

[CLINICAL PROCEDURE]**1. Cleaning**

Roughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.

2. Shade Selection

Prior to isolation of tooth, select the appropriate shade of ESTELITE UNIVERSAL FLOW using shade guide provided for ESTELITE UNIVERSAL FLOW.

3. Isolation

A rubber dam is the preferred method of isolation.

4. Cavity Preparation

Prepare the cavity. In case where no cavity preparation has been made (caries-free cervical defects), clean the tooth surface with a rubber cup and a fluoride-free cleaning paste. Rinse thoroughly with water.

- In the case of cervical caries, clean the tooth surface with a rubber cup and a fluoride-free cleaning paste. Rinse thoroughly with water; air dry thoroughly and treat with a silane coupling agent according to manufacturer's instructions.

5. Pulp Protection

Careful consideration should be applied if the cavity is in close proximity to the pulp. DO NOT USE LUGENIUM-BASED MATERIALS to protect the pulp as these materials will inhibit bonding of ESTELITE UNIVERSAL FLOW. ESTELITE UNIVERSAL FLOW is not indicated for direct pulp capping.

6. Bonding System

Apply a bonding system according to its manufacturer's instructions.

- Use the bonding/composite regimen to be used to condition the surface of porcelain/composite with a silane coupling agent before performing the bonding procedure.

7. Dispensing**7-1. PLT (Metal tip)**

Please refer to **SPECIAL NOTES FOR THE USE OF PLT** before use.

- Load the PLT into a dispenser that fits the PLT.

- Remove the PLT.

7-2. L-SYRINGE (High, Medium and Super Low)

- Hold the nozzle and turn the cap counter-clockwise for removal. [1]

- Hold the nozzle and place the Dispensing Tip (metal tip) onto it. Turn the Dispensing Tip clockwise until it is securely locked. [2]

7-3. Dispensing

- Accessories and Dispensing Tip are exclusively designed for the L-SYRINGE of ESTELITE UNIVERSAL FLOW. Use the attached Dispensing Tips (metal tips).

- Dispensing Tips (TOKUYAMA Dispensing Tip) are also sold separately.

- Use uncontaminated gloves when handling the syringe and attaching the Dispensing Tip.

8. Filling and contouring

- Accessories and Dispensing Tip are exclusively filled from the Dispensing Tip,PLT, or indirectly filled with an instrument after extracting the part on the mixing pad. Instruments should not exceed the indicated curing depth at a time (please refer to the aforementioned table).

- DO NOT apply excessive force to extrude the material from the syringe or PLT immediately after removal to prevent damage to the restorative.

- Do not use ESTELITE UNIVERSAL FLOW to avoid incomplete cure or entrainment of air bubbles.

- In case of syringe, to avoid cross infection, after extruding the paste, remove and discard the Dispensing Tip. Turn the nozzle with gaze and replace the cap immediately.

9. Curing

- Use the appropriate curing light for at least the indicated time (please refer to the aforementioned table).

- Apply the curing light for a minimum of 10 seconds.

- Turn off the curing light after the curing cycle is completed.

10. Finishing and Polishing

- Opalescent shades (OPA2, OPA3, OPA4) have adequate opacity designed for blocking out the dark shade that comes through the oral cavity (Class III and IV restoration). The opalescent shades also are designed for maximum light stain or reconstructing a highly opaque tooth. Please check with your local distributor for available shades.

11. Product Disposal

- Opalescent shades (OPA2, OPA3, OPA4) have adequate opacity designed for blocking out the dark shade that comes through the oral cavity (Class III and IV restoration). The opalescent shades also are designed for maximum light stain or reconstructing a highly opaque tooth. Please check with your local distributor for available shades.

12. Storage

- Store ESTELITE UNIVERSAL FLOW at temperatures between 0 - 25°C (32 - 77°F).

- Avoid direct exposure to light and heat.

- DO NOT USE ESTELITE UNIVERSAL FLOW after the indicated date of expiration on the syringe or PLT package.

13. Clinical Procedure

To avoid disposal of excess ESTELITE UNIVERSAL FLOW, extrude unused portion from SYRINGE or PLT and light-cure before disposal.

14. Clinical Procedure

1. Cleaning

Roughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.

2. Shade Selection

Prior to isolation of tooth, select the appropriate shade of ESTELITE UNIVERSAL FLOW using shade guide provided for ESTELITE UNIVERSAL FLOW.

3. Isolation

A rubber dam is the preferred method of isolation.

4. Cavity Preparation

Prepare the cavity. In case where no cavity preparation has been made (caries-free cervical defects), clean the tooth surface with a rubber cup and a fluoride-free cleaning paste. Rinse thoroughly with water.

- In the case of cervical caries, clean the tooth surface with a rubber cup and a fluoride-free cleaning paste. Rinse thoroughly with water; air dry thoroughly and treat with a silane coupling agent according to manufacturer's instructions.

5. Pulp Protection

Careful consideration should be applied if the cavity is in close proximity to the pulp. DO NOT USE LUGENIUM-BASED MATERIALS to protect the pulp as these materials will inhibit bonding of ESTELITE UNIVERSAL FLOW. ESTELITE UNIVERSAL FLOW is not indicated for direct pulp capping.

6. Bonding System

Apply a bonding system according to its manufacturer's instructions.

- Use the bonding/composite regimen to be used to condition the surface of porcelain/composite with a silane coupling agent before performing the bonding procedure.

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