6264182201 BPE 1A ENGLISH БЪЛГАРСКИ FRANÇAIS ΕΛΛΗΝΙΚΑ
ITALIANO PORTUGUES ROMANA ESPAÑOL TURKCE

One Component Self-Etching Light-Cured Dental Adhesive

TOKUYAMA BOND FORCE II

TOKUYAMA BOND FORCE II Pen

Last updated 2022-02-28 Rev.B

ENGLISH

Read all information, precautions and notes before using.

■PRODUCT DESCRIPTION AND GENERAL INFORMATION

1. TOKUYAMA BOND FORCE II is a one-component, one-coat application, self-etching, light-cured, dental adhesive system with excellent handling and adhesive properties and exceptional marginal integrity toward cut/cut enamel and dentin when used in combination with light- or dual-cured composite materials.
2. A light-curing unit having the camphorquinone (CQ) wavelength range (peak: 470nm, spectrum: 400 to 500nm) can be used for curing TOKUYAMA BOND FORCE II.
3. TOKUYAMA BOND FORCE II contains Phosphoric acid monomer, Bisphenol A di(2-hydroxy propyl) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate, 2-Hydroxyethyl methacrylate (HEMA), Camphorquinone, alcohol and purified water. The pH level immediately after dispensing is approximately 2.5.
4. TOKUYAMA BOND FORCE II is available in three convenient dispensing systems, Bottle, Pen-Type and Unit Dose. Please check with your local distributor for availability.

■INDICATIONS

- Bonding of light- or dual-cured composite material to: - cut/cut enamel, - cut/cut dentin, - finished porcelain/composite repair.
- CONTRAINDICATIONS
- TOKUYAMA BOND FORCE II contains methacrylic monomers, organic solvents and acids. DO NOT use on patients allergic to or hypersensitive to methacrylic monomers, related organic solvents and acids.
- PRECAUTIONS
- 1) DO NOT use TOKUYAMA BOND FORCE II for any purpose other than those listed in these instructions. Use TOKUYAMA BOND FORCE II only as directed herein.
- 2) TOKUYAMA BOND FORCE II is designed for safe and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
- 3) DO NOT use TOKUYAMA BOND FORCE II if the safety seals are broken or appear to have been tampered with.
- 4) If TOKUYAMA BOND FORCE II causes an allergic reaction or oversensitivity, discontinue its use immediately.
- 5) Use examination gloves (plastic, vinyl or latex) at all times when handling TOKUYAMA BOND FORCE II to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If TOKUYAMA BOND FORCE II comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- 6) Avoid contact with TOKUYAMA BOND FORCE II with eyes, mucosal membrane, skin and clothing.
- If TOKUYAMA BOND FORCE II comes in contact with eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
- If TOKUYAMA BOND FORCE II comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration is completed. Affected areas may whiten from protein coagulation, but such whitening should disappear within 24 hours. If such whitening does not disappear within 24 hours, immediately contact a physician, and the patient should be so advised.
- If TOKUYAMA BOND FORCE II comes in contact with the skin or clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze.
- Instruct the patient to rinse his mouth immediately after treatment.
- 7) TOKUYAMA BOND FORCE II should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- 8) To avoid the unintentional ingestion of TOKUYAMA BOND FORCE II, do not leave it unattended within the reach of patients and children.
- 9) DO NOT expose TOKUYAMA BOND FORCE II or the vapor to open flame because TOKUYAMA BOND FORCE II is flammable.
- 10) DO NOT avoid cross or lowering bond strength, DO NOT reuse the disposable applicator. Clean the dispensing unit thoroughly with alcohol.
- 11) DO NOT use TOKUYAMA BOND FORCE II directly to the cavity being in close proximity to the pulp. Pulp protection with glass ionomer lining or calcium hydroxide is recommended.
- 12) When using a light-curing unit, protective eyewear, glasses or goggles should be worn at all times. Be aware that when a restoration is being placed on occlusal or buccal cusp (clenching, grinding or tapping), the repaired restoration may chip again.
- 13) If not in use for a prolonged period of time, store in a cool dark place.
- 14) TOKUYAMA BOND FORCE II is not compatible with other brands of primers or adhesives.
- 15) TOKUYAMA BOND FORCE II should be brought to room temperature before using if it is stored under refrigeration.
- 16) Do not use the Pen dispensing system.
- DO NOT push the button when the cap is closed.
- DO NOT push the button when the nozzle is facing up.
- Avoid holding the Pen vertically while dispensing.
- DO NOT use the Pen directly in the oral cavity.
- DO NOT dismantle the Pen.

■PRECAUTIONS FOR MEDICATIONS AND MATERIALS

- 1) Some materials and medications (hemostatic materials) inhibit the adhesion of TOKUYAMA BOND FORCE II for an extended period even after careful meticulous cleansing with water. DO NOT USE products which contain: - eugenol, - diamine silver thioether [molecular formula: Ag(NH₂)₂], - phenols such as paraethoxyphenol, guaiacol, phenol, - aluminum chloride, - ferric sulfate, - aluminum sulfate, - epierythrine.
- 2) TOKUYAMA BOND FORCE II does not adhere to tooth structure immediately after applications of materials which contain: - hydrogen peroxide (oxygenol), - sodium hypochlorite.
- 3) We advise avoiding the use of these materials on the day of cementation, however these materials can be used for post-canal treatment as the inhibition caused by these materials usually evaporates within five days.
- STORAGE
- 1) Store TOKUYAMA BOND FORCE II at a temperature between 0 to 25°C (32 to 77°F). Storage under inappropriate condition will accelerate the degradation of TOKUYAMA BOND FORCE II.
- 2) If not in use for a prolonged period of time, store in a cool dark place.
- 3) Keep away from heat, direct sunlight, sparks and open flames.
- 4) DO NOT use after the indicated expiration date on the bottle/package.
- DISPOSAL
- 1) Unused TOKUYAMA BOND FORCE II is to be absorbed into an inert absorbent material such as gauze or cotton, and disposed of in accordance with local regulations.

■CLINICAL PROCEDURES

1. **Cleaning**
Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.
2. **Isolation**
The rubber dam is the preferred method of isolation.
3. **Cavity preparation**
Prepare the cavity and rinse with water. Add bevies to the enamel margins of anterior preparations (class III, IV, V), as well as chamfers to the margins of posterior preparations (class I, II). Bevels and chamfers assist in retaining cementation between the cavity margins and the restoration, enhancing both esthetics and retention.
4. **Etching**
- If there is an adherent surface including uncut enamel, apply etching agent to the uncut enamel.
- In case of porcelain/composite repairs, roughen the surface with a bur or a diamond point to prepare the area for adhesion; apply phosphoric acid for cleaning; rinse thoroughly with water; air dry thoroughly and treat with a silane coupling reagent according to its manufacturer's instructions.
5. **Drying**
Dry the cavity by blotting with an air syringe.
- DO NOT desiccate the vital tooth. Desiccation can lead to post-operative sensitivity.
- The substances listed below inhibit curing and should be removed from the tooth surface by thoroughly cleaning the tooth surface with alcohol, citric acid, or the application of phosphoric acid for 2 to 3 seconds before application.- 1) Oil mist from handpiece,- 2) Saliva, blood and exudates.

■Pulp Protection

Glass ionomer lining or calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT USE EUGENOL BASED MATERIALS to protect the pulp as these materials will inhibit the curing of TOKUYAMA BOND FORCE II.

■6. Dispensing

- 6-1. Unit Dose
Test open the Unit Dose container and dry the disposable applicator.
- When opening the cap, make sure to verify the top and bottom of the container.
- To avoid cross infection, do not reuse the material on other patients.

6-2. Bottle

Open the Bottle by removing the cap and dispense one or two drops of adhesive into the dispensing well. Immediately after dispensing, tightly replace the Bottle cap.

- Be sure to hold the Bottle of TOKUYAMA BOND FORCE II vertically when dispensing. Do not hold the Bottle diagonally or horizontally while dispensing; this could cause the adhesive to flow back into the nozzle and become contaminated.


- Wipe off excess adhesive from the tip of the nozzle prior to closing.

6-3. Pen

Open the Pen by removing the cap when the cap is facing up and dispense one or two drops of adhesive while holding by holding the Pen vertically and pushing on button repeatedly until adhesive starts to flow (or 2 to 3 pushes). * This depends on how the button is pushed and the position that the Pen is being held. Immediately after dispensing, tightly replace the Pen cap.

- Be sure to hold the Pen vertically when dispensing. Do not hold the Pen diagonally or horizontally while dispensing; this could cause the adhesive to flow back into the nozzle and become contaminated.

- Wipe off excess adhesive from the tip of the nozzle prior to closing.

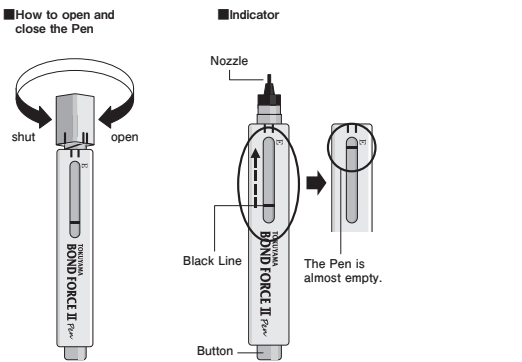The Indicator allows users to monitor the remaining amount of adhesive. Black Line moves closer to the E as the amount of material decreases. When the line nears "F", this indicates that the Pen is almost empty. This is not an exact reading but only an indication of the remaining material.

7. Application

Using the disposable applicator, apply TOKUYAMA BOND FORCE II on the surface to be bonded and wait 10 seconds or more.

- Ensure TOKUYAMA BOND FORCE II covers all surfaces where the composite resin will be applied.

- Protect the dispensed adhesive and the inserted applicator from ambient light before the application.

Unit Dose: keep the applicator in the Unit Dose container.

Bottle, Pen: use a light blocking plate.

- Complete the application within 5 minutes after dispensing because TOKUYAMA BOND FORCE II contains a volatile alcohol.

- Be careful not to allow excess TOKUYAMA BOND FORCE II to flow into the subgingival. If TOKUYAMA BOND FORCE II flows into the subgingival, thoroughly rinse and dry the area.

- If the adherent surface include ceramics, porcelain or composite, prepare the surface according to the appropriate procedures prior to the application of TOKUYAMA BOND FORCE II.

8. Air Dry

Using an oil-free air syringe, apply mild air continuously to the TOKUYAMA BOND FORCE II surface until the surface is completely dry. Do not approach too close to the adhesive.

- If accidental spattering occurs, it may cause the tissue to whiten or possible allergic reaction.

- If the adhesive is still on the tooth floor or the crown surface area, it is too thick to air-dry, blot the excess with a new disposable applicator before mild air application.

- If saliva, blood, or other fluids contaminate the applied adhesive, thoroughly rinse the cavity with water, dry and re-apply new adhesive.

- Do not rinse the applied adhesive with water except in unintentional contamination.

10. Light-cure Composite

After cementation of TOKUYAMA BOND FORCE II according to its manufacturer's instructions. The composite resin must be finished and polished.

- When self-cured composite resins are placed into a cavity, the first increment must be light-cured using a light-curing unit.

- Do not use self-cured composite resins here because the Phosphoric acid monomer contained in TOKUYAMA BOND FORCE II may interfere with the curing of the self-cured resins, leading to their premature detachment.

- If TOKUYAMA BOND FORCE II comes in contact with the skin or clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze.

- Instruct the patient to rinse his mouth immediately after treatment.

7) TOKUYAMA BOND FORCE II should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.

8) To avoid the unintentional ingestion of TOKUYAMA BOND FORCE II, do not leave it unattended within the reach of patients and children.

9) DO NOT expose TOKUYAMA BOND FORCE II or the vapor to open flame because TOKUYAMA BOND FORCE II is flammable.

10) DO NOT avoid cross or lowering bond strength, DO NOT reuse the disposable applicator. Clean the dispensing unit thoroughly with alcohol.

11) DO NOT use TOKUYAMA BOND FORCE II directly to the cavity being in close proximity to the pulp. Pulp protection with glass ionomer lining or calcium hydroxide is recommended.

12) When using a light-curing unit, protective eyewear, glasses or goggles should be worn at all times. Be aware that when a restoration is being placed on occlusal or buccal cusp (clenching, grinding or tapping), the repaired restoration may chip again.

13) If not in use for a prolonged period of time, store in a cool dark place.

14) TOKUYAMA BOND FORCE II is not compatible with other brands of primers or adhesives.

15) TOKUYAMA BOND FORCE II should be brought to room temperature before using if it is stored under refrigeration.

16) Do not use the Pen dispensing system.

- DO NOT push the button when the cap is closed.

- DO NOT push the button when the nozzle is facing up.

- Avoid holding the Pen vertically while dispensing.

- DO NOT use the Pen directly in the oral cavity.

- DO NOT dismantle the Pen.

■PRECAUTIONS FOR MEDICATIONS AND MATERIALS

- 1) Some materials and medications (hemostatic materials) inhibit the adhesion of TOKUYAMA BOND FORCE II for an extended period even after careful meticulous cleansing with water. DO NOT USE products which contain: - eugenol, - diamine silver thioether [molecular formula: Ag(NH₂)₂], - phenols such as paraethoxyphenol, guaiacol, phenol, - aluminum chloride, - ferric sulfate, - aluminum sulfate, - epierythrine.
- 2) TOKUYAMA BOND FORCE II does not adhere to tooth structure immediately after applications of materials which contain: - hydrogen peroxide (oxygenol), - sodium hypochlorite.
- 3) We advise avoiding the use of these materials on the day of cementation, however these materials can be used for post-canal treatment as the inhibition caused by these materials usually evaporates within five days.

■STORAGE

- 1) Store TOKUYAMA BOND FORCE II at a temperature between 0 to 25°C (32 to 77°F). Storage under inappropriate condition will accelerate the degradation of TOKUYAMA BOND FORCE II.
- 2) If not in use for a prolonged period of time, store in a cool dark place.
- 3) Keep away from heat, direct sunlight, sparks and open flames.
- 4) DO NOT use after the indicated expiration date on the bottle/package.

■DISPOSAL

Unused TOKUYAMA BOND FORCE II is to be absorbed into an inert absorbent material such as gauze or cotton, and disposed of in accordance with local regulations.

■CLINICAL PROCEDURES

1. **Cleaning**
Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.
2. **Isolation**
The rubber dam is the preferred method of isolation.
3. **Cavity preparation**
Prepare the cavity and rinse with water. Add bevies to the enamel margins of anterior preparations (class III, IV, V), as well as chamfers to the margins of posterior preparations (class I, II). Bevels and chamfers assist in retaining cementation between the cavity margins and the restoration, enhancing both esthetics and retention.
4. **Etching**
- If there is an adherent surface including uncut enamel, apply etching agent to the uncut enamel.
- In case of porcelain/composite repairs, roughen the surface with a bur or a diamond point to prepare the area for adhesion; apply phosphoric acid for cleaning; rinse thoroughly with water; air dry thoroughly and treat with a silane coupling reagent according to its manufacturer's instructions.
5. **Drying**
Dry the cavity by blotting with an air syringe.
- DO NOT desiccate the vital tooth. Desiccation can lead to post-operative sensitivity.
- The substances listed below inhibit curing and should be removed from the tooth surface by thoroughly cleaning the tooth surface with alcohol, citric acid, or the application of phosphoric acid for 2 to 3 seconds before application.- 1) Oil mist from handpiece,- 2) Saliva, blood and exudates.

■Pulp Protection

Glass ionomer lining or calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT USE EUGENOL BASED MATERIALS to protect the pulp as these materials will inhibit the curing of TOKUYAMA BOND FORCE II.

■6. Dispensing

- 6-1. Unit Dose
Test open the Unit Dose container and dry the disposable applicator.
- When opening the cap, make sure to verify the top and bottom of the container.
- To avoid cross infection, do not reuse the material on other patients.

6-2. Bottle

Open the Bottle by removing the cap and dispense one or two drops of adhesive into the dispensing well. Immediately after dispensing, tightly replace the Bottle cap.

- Be sure to hold the Bottle of TOKUYAMA BOND FORCE II vertically when dispensing. Do not hold the Bottle diagonally or horizontally while dispensing; this could cause the adhesive to flow back into the nozzle and become contaminated.

- Wipe off excess adhesive from the tip of the nozzle prior to closing.

6-3. Pen

Open the Pen by removing the cap when the cap is facing up and dispense one or two drops of adhesive while holding by holding the Pen vertically and pushing on button repeatedly until adhesive starts to flow (or 2 to 3 pushes). * This depends on how the button is pushed and the position that the Pen is being held. Immediately after dispensing, tightly replace the Pen cap.

- Be sure to hold the Pen vertically when dispensing. Do not hold the Pen diagonally or horizontally while dispensing; this could cause the adhesive to flow back into the nozzle and become contaminated.

- Wipe off excess adhesive from the tip of the nozzle prior to closing.

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