8254412201 BULK IA
 ENGLISH
 БЪЛГАРСКИ
 FRANÇAIS
 ΕΛΛΗΝΙΚΑ
 ITALIANO
 PORTUGUES
 ROMÂNĂ
 ESPAÑOL
 TÜRKÇE

Resin-based Dental Restorative Material

ESTELITE BULK FILL Flow SYRINGE

ESTELITE BULK FILL Flow PLT

Last updated 2022-02-28 Rev.E

ENGLISH

Read all information, precautions and notes before using.

■PRODUCT DESCRIPTION AND GENERAL INFORMATION

- ESTELITE BULK FILL Flow is a low viscosity, light-cured, radiopaque composite resin. Its low stress flowable material can be placed 4mm increments.
- ESTELITE BULK FILL Flow incorporates Radical-Amplified Photopolymerization initiator technology (RAP technology). RAP technology facilitates a shortened light curing time and ample working time.
- ESTELITE BULK FILL Flow contains 70% by weight (56% by volume) of spherical silica-zirconia filler and composite filler. The composite filler offers reduced polymerization shrinkage stress due to its round shape. The spherical filler contained in ESTELITE BULK FILL Flow (mean particle size: 200nm, particle size range: 10 to 300nm) facilitates excellent gloss retention and wear resistance. ESTELITE BULK FILL Flow contains Bisphenol A di(2-hydroxy propxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate (TEGDMA), Bisphenol A polyethoxy methacrylate (Bis-MPEPP), Mequiol, Dibutyl hydroxyl toluene and UV absorber.
- ESTELITE BULK FILL Flow is provided either in SYRINGE or Pre Loaded Tip (PLT).

■SHADE

U (universal), A1, A2, A3, B1

Please check with your local distributor for available shades.

■INDICATIONS

- Direct anterior and posterior restorations
- Cavity lining
- Blocking out cavity undercuts before fabricating indirect restorations
- Repair of porcelain/composite

■CONTRAINDICATIONS

ESTELITE BULK FILL Flow contains methacrylic monomers. DO NOT use ESTELITE BULK FILL Flow for patients allergic to or hypersensitive to methacrylic and related monomers or to any of the other ingredients.

■PRECAUTIONS

- DO NOT use ESTELITE BULK FILL Flow for any purposes other than those listed in these instructions. Use ESTELITE BULK FILL Flow only as directed herein.
- ESTELITE BULK FILL Flow is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor use by non-dental care professionals.
- DO NOT use ESTELITE BULK FILL Flow if the safety seals are broken or appear to have been tampered with.
- ESTELITE BULK FILL Flow causes an allergic reaction or hypersensitivity, discontinue its use immediately.
 - Use examination gloves (plastic, vinyl, or latex) at all times when handling ESTELITE BULK FILL Flow. Use the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If ESTELITE BULK FILL Flow comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- Avoid contact of ESTELITE BULK FILL Flow with eyes, mucosal membrane, skin and clothing.
 - If ESTELITE BULK FILL Flow comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If ESTELITE BULK FILL Flow comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration is completed.
 - If ESTELITE BULK FILL Flow comes into contact with the skin or clothing, immediately saturate the area with an alcohol soaked cotton swab or gauze.
 - Instruct the patient to rinse his mouth immediately after treatment.
- ESTELITE BULK FILL Flow should not be ingested or aspirated. Ingestion or aspiration causes serious injury.
- Do not allow the unintentional ingestion of ESTELITE BULK FILL Flow, do not leave it ununservised within the reach of patients and children.
- Clean the placement instruments and brushes with alcohol after use.
- When using a light-curing unit, protective eye shields, glasses or goggles should be worn at all times.
- ESTELITE BULK FILL Flow is available in semi-translucent shades. If applied in cavity Class III or IV direct restorations, the cured ESTELITE BULK FILL Flow may appear dark. Avoid using ESTELITE BULK FILL Flow for the direct restoration of Class III or IV when applying is permanent.
- Direct-cure ESTELITE BULK FILL Flow extra orally to confirm the time required for complete hardening of ESTELITE BULK FILL Flow with your light-curing unit prior to doing the restoration.
- This product is designed to be used at room temperature (18 - 30°C (62 - 84°F)). Allow product to reach room temperature prior to use. Cold material may be difficult to extrude.

■CURING TIME AND DEPTH

ESTELITE BULK FILL Flow enables a 4mm depth of cure (U shade) by light-curing for 10 seconds with Halogen or LED light-curing unit (Light intensity: 800mW/cm² or more wavelength: 400-500nm). The depth of cure was determined based on test results performed in accordance with section 7.10 "Depth of cure" of ISO 4049.

■SPECIAL NOTES FOR THE USE OF PLT

- PLTs are designed for single patient use only. To avoid cross infection, Do not re-cap and/or re-use the PLT tip once material has been dispensed for that patient.
- Dispensers are not provided for ESTELITE BULK FILL Flow. Use a dispenser that fits the PLT of ESTELITE BULK FILL Flow. For proper use and proper disinfection, see manufacturer's instructions.
- Use light, controlled pressure to prevent any continued extrusion of material following pressure release.

■STORAGE

- Store ESTELITE BULK FILL Flow at a temperature between 0 - 25°C (32 - 77°F). Shelf life at this temperature is 3 years.
- AVOID direct exposure to light and heat.
- DO NOT use ESTELITE BULK FILL Flow after the indicated date of expiration on the syringe or PLT package.

■DISPOSAL

To safely dispose of excess ESTELITE BULK FILL Flow, extrude unused portion from SYRINGE or PLT and light-cure before disposal.

■CLINICAL PROCEDURE

- Cleaning**

Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.
- Shade Selection**

Prior to isolation of tooth, select the appropriate shade of ESTELITE BULK FILL Flow using shade card provided for ESTELITE BULK FILL Flow.
- Isolation**


A rubber dam is the preferred method of isolation.
- Cavity Preparation**

Prepare the cavity and rinse with water.

 - In the case of porcelain/composite repairs, roughen the surface with a bur or a diamond point to prepare the area for adhesion; apply phosphoric acid etch for cleaning; rinse thoroughly with water; air dry thoroughly and treat with a silane coupling agent according to manufacturer's instructions.
- Pulp Protection**

Calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT USE EUGENOL-BASED MATERIALS SO protect the pulp as these materials will inhibit curing ESTELITE BULK FILL Flow. ESTELITE BULK FILL Flow is not indicated for direct pulp capping.
- Bonding system**

Apply a bonding system according to its manufacturer's instructions.
- Dispensing**
 - PLT
 - Press read **SPECIAL NOTES FOR THE USE OF PLT** before use.
 - Load the **PLT** into a dispenser that fits the PLT.
 - Remove the PLT cap.
 - SYRINGE
 - Hold the nozzle and turn the cap counterclockwise for removal. [1]
 - Hold the nozzle and place the Dispensing Tip onto it. Turn the Dispensing Tip clockwise until it is securely locked. [2]

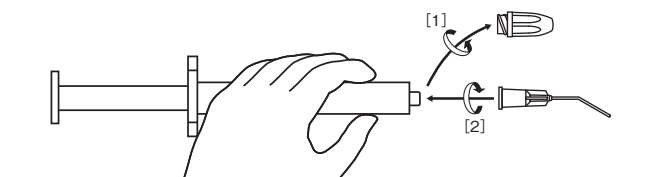- Accessories and Dispensing Tip are exclusively designed for ESTELITE BULK FILL Flow. Avoid using Dispensing Tips from other brands with ESTELITE BULK FILL Flow. - Dispensing Tips are also sold separately. - Use uncontaminated gloves when handling the syringe and attaching the Dispensing Tip.

8. Filling and contouring

- Cap must be directly filled from the Dispensing Tip/PLT. Increments should not exceed 4mm at a time.
- DO NOT apply excessive force to extrude the material from the syringe or PLT immediately after removal from the refrigerator.
- DO NOT mix ESTELITE BULK FILL Flow with other brands of resin composite or other shades of ESTELITE BULK FILL Flow. Flow to avoid incomplete cure or entrapment of air bubbles.
- In case of syringe, to avoid cross infection, after extruding the paste, remove and discard the Dispensing Tip, wipe the nozzle with gauze and replace the cap immediately.
 - This product is provided non-sterile and is not intended to be sterilized by the user.

9. Curing

Light-cure each increment for at least the indicated time (please see the curing recommendations below). Light-cure each layer (up to 4mm increment) using Halogen light-curing or LED light-curing unit with a wavelength of 400-500 nm. Be sure the light-curing unit with an intensity of 800mW/cm² or more. Hold the light guide tip as close to the restorative as possible during light exposure.

CURING RECOMMENDATIONS

Shade	Increment Depth	Light Output	Curing Time
All shades	4mm	≥800mW/cm ²	10 sec.

- If other brands of composite resins are layered over the cured composite, follow their instructions.

10. Finishing

Shape and polish the restoration. For finishing, use finishing discs and/or fine finishing diamond points. Use metal finishing strips or vinyl polishing strips for proximal surfaces. For polishing, polish with rubber points on any suitable polishing tooth. For final polishing, use felt discs or cotton wheels with polishing paste, or suitable polishing tool.

- Instruct the patient adequately on the maintenance of the restoration after treatment.

CAUTION: Federal law restricts this device to sale by or on the order of a dental professional.

The user and/or patient should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the dental professional to ensure the product is suitable for application before use.

Specifications are subject to change without notice. When the product specification changes, the instructions and precautions may change also.

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