Instructions for Use

Resin-based Dental Restorative Material

ESTELITE ΣQUICK 
 CE 1639

ESTELITE ΣQUICK SHADE GUIDE

One Component Self-Etching Light-Cured Dental Adhesive

TOKUYAMA BOND FORCE II
 CE 1639

Tokuyama Dental Corporation

38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016, Japan

Tel: +81-3-3835-7201

URL: <https://tokuyama-dental.com>

ENGLISH	1	БЪЛГАРСКИ	4	ČESKY	7	DEUTSCH	10
DANSK	14	EESTI KEEL	17	ESPAÑOL	20	SUOMI	24
FRANÇAIS	27	ΕΛΛΗΝΙΚΑ	30	HRVATSKI	34	MAGYAR	37
ITALIANO	40	LIETUVIŲ K.	43	LATVISKI	47	NEDERLANDS	50
NORSK	53	POLSKI	56	PORTUGUÊS	60	ROMÂNĂ	63
SVENSKA	66	SLOVENČINA	69	SLOVENŠČINA	72	TÜRKÇE	76

This product is certified as a medical device under the legislation of the European Union on medical devices by SGS CE1639, exclusively for the indication(s) shown in this IFU. Other non-medical uses ascribed to this product are not within the scope of CE certification, and users should be aware product performance and/or safety has not been evaluated by SGS for those purposes.

Resin-based Dental Restorative Material

ESTELITE Σ QUICK

QUICK-CURE

Read all information, precautions and notes before using.

■PRODUCT DESCRIPTION AND GENERAL INFORMATION

- ESTELITE Σ QUICK is a light-cured, radiopaque, submicron filled composite resin for use in anterior and posterior restorations and is indicated for all carious classes. It has radiopacity equal to or greater than 1mm aluminum, and is classified as a Type 1 and Class 2 (Group 1) material by ISO 4049. ESTELITE Σ QUICK contains 82% by weight (71% by volume) of silica-zirconia filler and composite filler. A high filler load offers decreased polymerization shrinkage. All inorganic filler contained in ESTELITE Σ QUICK is a spherical submicron filler (mean particle size: 0.2 μm, particle size range: 0.1 to 0.3μm) that is designed to deliver excellent gloss retention and wear resistance. The monomer matrix contains Bis-GMA and Triethylene glycol dimethacrylate.
- ESTELITE Σ QUICK implements Radical-Amplified Photopolymerization initiator technology (RAP technology). Curing time for ESTELITE Σ QUICK can be shortened by one-third using a light curing unit with a wavelength range of 400 to 500nm (peak: 470nm) compared to our conventional resin composite. Please see the table depicting the relationship between curing time and increment depth (■INDICATIONS FOR FILLING AND CURING).
- RAP technology facilitates a longer working time for ESTELITE Σ QUICK.
- ESTELITE Σ QUICK is provided either in SYRINGE or Pre Loaded Tip(PLT).

■SHADE

- A1, A2, A3, A3.5, A4, A5, B1, B2, B3, B4, C1, C2, C3, OA1, OA2, OA3, OPA2, BW (Bleach White), WE (White Enamel) and CE (Clear Enamel/Incisal).
- Opalescent shades (OA1, OA2, OA3) have adequate opacity designed for blocking out the dark shine that comes through the oral cavity (Class III and Class IV restoration). The opalescent shades also serve as a dentin layer for the multi-shade layering technique. Shades OA1, OA2 and BW closely match those of primary teeth. However, they are not designed for masking metal or a darkly stained area such as tetracycline stained.
 - OPA2 is designed for masking slight stain or reconstructing a highly opaque tooth (use as a thin layer at the lingual cavity wall of class III and IV).
 - Please check with your local distributor for available shades.

■INDICATIONS

- Direct anterior and posterior restorations including the occlusal surface and all carious classes
- Direct bonded composite veneer
- Diastema closure
- Repair of porcelain/composite

■CONTRAINDICATIONS

ESTELITE Σ QUICK contains methacrylic monomers. DO NOT use ESTELITE Σ QUICK for patients allergic to or hypersensitive to methacrylic and related monomers.

■PRECAUTIONS

- DO NOT use ESTELITE Σ QUICK for any purpose other than those listed in these instructions. Use ESTELITE Σ QUICK only as directed herein.
- ESTELITE Σ QUICK is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
- DO NOT use ESTELITE Σ QUICK if the safety seals are broken or appear to have been tampered with.
- If ESTELITE Σ QUICK causes an allergic reaction or oversensitivity, discontinue its use immediately.
- Use examination gloves (plastic, vinyl, or latex) at all times when handling ESTELITE Σ QUICK to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If ESTELITE Σ QUICK comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- Avoid contact of ESTELITE Σ QUICK with eyes, mucosal membrane, skin and clothing.
 - If ESTELITE Σ QUICK comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If ESTELITE Σ QUICK comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration.
 - If ESTELITE Σ QUICK comes into contact with the skin or clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze.
 - Instruct the patient to rinse his mouth immediately after treatment.
- ESTELITE Σ QUICK should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- To avoid the unintentional ingestion of ESTELITE Σ QUICK, do not leave it unsupervised within the reach of patients and children.

- Clean the placement instruments with alcohol after use.
- When using a light-curing unit, protective eye shields, glasses or goggles should be worn.

■INDICATIONS FOR FILLING AND CURING

Be sure to light-cure ESTELITE Σ QUICK extraorally and check the time needed for complete hardening of ESTELITE Σ QUICK with your light curing unit before performing the bonding procedure. The following table illustrates the relationship between curing time and increment depth.

Relationship between curing time and increment depth:

Light type	Intensity (mW/cm ²)	Curing time (seconds)	Increment depth (mm) ⁽¹⁾	
			A1, A2, A3, A3.5, B1, B2, B3, C1, C2, OA1, OA2, CE(Inc.), BW, WE	A4, A5, B4, C3, OA3, OPA2
Halogen	400	10	1.9	1.6
		20	2.2	1.9
	600	10	2.0	1.7
		20	2.5	2.0
	800	5	1.8	1.6
		10	2.2	1.9
LED	260	10	1.7	1.4
		20	2.0	1.7
	900	5	1.7	1.5
		10	2.0	1.8
Plasma Arc	950	3	1.9	1.5

- Increment depth was determined on the basis of test results performed in accordance with section 7.10 "depth of cure" of ISO4049.

■SPECIAL NOTES FOR THE USE OF PLT

- PLTs are designed for single patient use only. To avoid cross infection, Do not re-cap and/or re-use the PLT tip once material has been dispensed for that patient.
- Dispensers are not provided for ESTELITE Σ QUICK. Use a dispenser that fits the PLT of ESTELITE Σ QUICK. For proper use and proper disinfection, see manufacturer's instructions.
- Apply the composite at normal room temperature (18 - 30°C / 62 - 84°F). Material that is too cold might be difficult to extrude.
- Use light, controlled pressure to prevent any continued extrusion of material following pressure release.

■STORAGE

- Store ESTELITE Σ QUICK at a temperature between 0 - 25°C (32 - 77°F).
- Avoid direct exposure to light and heat.
- DO NOT use ESTELITE Σ QUICK after the indicated date of expiration on the syringe or PLT package.

■DISPOSAL

To safely dispose of excess ESTELITE Σ QUICK, extrude unused portion from SYRINGE or PLT and light-cure before disposal. Do not allow to enter drains, sewers or watercourses. Must not be disposed of together with household garbage. Follow local instructions for disposal. Refer to the packaging waste disposal guide shown on the end of this IFU.

■CLINICAL PROCEDURE

1. Cleaning

Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.

2. Shade Selection

- Select the appropriate shade of ESTELITE Σ QUICK prior to placing the rubber dam using the shade guide provided for ESTELITE Σ QUICK.
- Complete the shade selection within 5 minutes; desiccated teeth are lighter than wet or moist teeth.
 - Lightness (color value) is primary importance on shade selection.
 - When wavering in shade selection, select the lighter (high value) shade.
 - In case of whitened teeth select the shade a few weeks after the completion of whitening. Whitened teeth may become slight darker.

3. Isolation

A rubber dam is the preferred method to isolate the tooth from contamination.

4. Cavity Preparation

- Prepare the cavity and rinse with water. Add bevels to the enamel margins of anterior preparations (class III, IV, V), as well as chamfers to the margins of posterior preparations (class I, II) because bevels and chamfers assist in erasing demarcations between the cavity margins and restoration, thereby enhancing both esthetics and retention.
- Scalloped (undulant) bevel could be desirable depending on the esthetics and retention.
 - In case of porcelain/composite repairs, roughen the surface with a bur or a diamond point to prepare the area for adhesion; apply phosphoric acid etch for cleaning; rinse thoroughly with water; air dry thoroughly and treat with a silane coupling agent according to manufacturer's instructions.

5. Pulp Protection

Glass ionomer lining or calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT USE EUGENOL BASED MATERIALS to protect the pulp as these materials will inhibit curing of bonding agent and/or ESTELITE Σ QUICK.

6. Bonding system

Apply a bonding system according to manufacturer's instructions.

7. Dispensing

7-1. PLT

Please read ■ SPECIAL NOTES FOR THE USE OF PLT before use.

- Load the PLT into a dispenser that fits the PLT.
- Remove the PLT cap.
- Extrude the paste directly into cavity or indirectly on the mixing pad.

7-2. SYRINGE

Remove the syringe cap. Extrude the paste onto a mixing pad by turning the handle clockwise. Re-cap the syringe immediately after dispensing.

- DO NOT apply unnecessary force to the syringe immediately after removing from the refrigerator.

8. Filling and contouring

Fill the cavity incrementally. Increments should not exceed the indicated depth (please refer to the aforementioned table).

- DO NOT mix ESTELITE Σ QUICK with other brands of resin composite or other shades of ESTELITE Σ QUICK to avoid incomplete cure or voids.

9. Curing

Light-cure each increment for at least the indicated time (please see aforementioned table), keeping the curing light tip within a distance of 2 mm from the increment.

- If other brands of composite resins are layered over the cured composite, follow their instructions.

10. Finishing

Shape and polish the restoration. For finishing, use fine finishing diamond points. To erase the demarcation between the composite and the natural tooth rotate a 12 fluted carbide bur slowly along the margins without water. Use finishing metal strips or vinyl strips for proximal surfaces. For polishing, polish with rubber points or any suitable polishing tool, such as PoGo™ (DENTSPLY/Caulk), Sof-Lex™ (3M ESPE), Identoflex® HiLuster Dia Polishers (Kerr), or D-FINE™ Hybrid Diamond (COSMEDENT, Inc.).

* PoGo, Sof-Lex, Identoflex and D-FINE are not registered trademarks of Tokuyama Dental Corporation.

- Instruct the patient adequately on the maintenance of the restoration after treatment.

The user and/or patient should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

IMPORTANT NOTE: The manufacturer of ESTELITE Σ QUICK is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use.

Product specifications of ESTELITE Σ QUICK are subjected to change without notification. When the product specification change, the instruction and precautions may change also.

ESTELITE Σ QUICK SHADE GUIDE

NOTICE FOR USE OF THE ESTELITE Σ QUICK SHADE GUIDE

1. Keep the ESTELITE Σ QUICK Shade Guide away from direct sunlight to prevent a change in color.

2. Avoid contact between the ESTELITE Σ QUICK Shade Guide and bonding agents, etc., because the surface of the ESTELITE Σ QUICK Shade Guide will dissolve if it comes into contact with methacrylic monomers or solvents.

The ESTELITE Σ QUICK Shade Guide is composed of the following materials:

Shade piece: Polystyrene

Base plate: ABS (Acrylonitrile-Butadiene-Styrene) resin

3. Refer to the following table when cleaning the ESTELITE Σ QUICK Shade Guide.

	Cleaning method
Correct	- wash with soap and water - ultrasonic cleaning while immersed in water - immerse in a 3% glutaraldehyde solution - wipe with a cotton swab soaked in an alcohol (80%) solution
Incorrect	- autoclave¹⁾ - dry heat sterilization¹⁾ - immerse in an alcohol (80%) solution²⁾ - immerse in a sodium hypochlorite solution³⁾ - immerse in a hydrogen peroxide solution³⁾

1) Shade piece and base plate will melt.

2) Base plate will whiten and crack. Shade indications and the logo will fade out.

3) Shade piece might be decolorized.

One Component Self-Etching Light-Cured Dental Adhesive

TOKUYAMA BOND FORCE II

Read all information, precautions and notes before using.

■ PRODUCT DESCRIPTION AND GENERAL INFORMATION

1. TOKUYAMA BOND FORCE II is a one-component, one-coat application, self-etching, light-cured, dental adhesive system with excellent handling and adhesive properties and exceptional marginal integrity toward cut/uncut enamel and dentin when used in combination with light- or dual-cured composite materials.
2. A light-curing unit having the camphorquinone (CQ) wavelength range (peak: 470nm, spectrum: 400 to 500nm) can be used for curing TOKUYAMA BOND FORCE II.
3. TOKUYAMA BOND FORCE II contains Phosphoric acid monomer, Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate, 2-Hydroxyethyl methacrylate (HEMA), Camphorquinone, alcohol and purified water. The pH level immediately after dispensing is approximately 2.8.
4. TOKUYAMA BOND FORCE II is available in three convenient dispensing systems, Bottle, Pen-Type and Unit Dose. Please check with your local distributor for availability.

■ INDICATIONS

Bonding of light- or dual-cured composite material to:

- cut/uncut enamel,
- cut/uncut dentin,
- fractured porcelain/composite repair.

■ CONTRAINDICATIONS

TOKUYAMA BOND FORCE II contains methacrylic monomers, organic solvents and acids. DO NOT use TOKUYAMA BOND FORCE II for patients allergic to or hypersensitive to methacrylic monomers, related monomers, organic solvents and acids.

■ PRECAUTIONS

- 1) DO NOT use TOKUYAMA BOND FORCE II for any purpose other than those listed in these instructions. Use TOKUYAMA BOND FORCE II only as directed herein.
- 2) TOKUYAMA BOND FORCE II is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
- 3) DO NOT use TOKUYAMA BOND FORCE II if the safety seals are broken or appear to have been tampered with.
- 4) If TOKUYAMA BOND FORCE II causes an allergic reaction or oversensitivity, discontinue its use immediately.
- 5) Use examination gloves (plastic, vinyl or latex) at all times when handling TOKUYAMA BOND FORCE II to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If TOKUYAMA BOND FORCE II comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- 6) Avoid contact of TOKUYAMA BOND FORCE II with eyes, mucosal membrane, skin and clothing.
 - If TOKUYAMA BOND FORCE II comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If TOKUYAMA BOND FORCE II comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration is completed. Affected areas may whiten from protein coagulation, but such whitening should disappear within 24 hours. If such whitening does not disappear within 24 hours, immediately contact a physician, and the patient should be so advised.
 - If TOKUYAMA BOND FORCE II comes into contact with the skin or clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze.
 - Instruct the patient to rinse his mouth immediately after treatment.
- 7) TOKUYAMA BOND FORCE II should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- 8) To avoid the unintentional ingestion of TOKUYAMA BOND FORCE II, do not leave it unsupervised within the reach of patients and children.
- 9) DO NOT expose TOKUYAMA BOND FORCE II or the vapor to open flame because TOKUYAMA BOND FORCE II is flammable.
- 10) To avoid cross infection or lowering bond strength, DO NOT reuse the disposable applicator. Clean the dispensing well thoroughly with alcohol after each use.
- 11) DO NOT use TOKUYAMA BOND FORCE II directly to the cavity being in close proximity to the pulp. Pulp protection with glass ionomer lining or calcium hydroxide is recommended.
- 12) When using a light-curing unit, protective eye shields, glasses or goggles should be worn at all times.
- 13) Be aware that when a restoration is chipped from misaligned occlusion or bruxism (clenching, grinding or tapping), the repaired restoration may chip again.

- 14) DO NOT mix TOKUYAMA BOND FORCE II with other brands of primers or adhesives.
- 15) TOKUYAMA BOND FORCE II should be brought to room temperature before using if it is stored under refrigeration.
- 16) If using the Pen dispensing system,
 - DO NOT push the button when the cap is closed.
 - DO NOT push the button when the nozzle is facing up.
 - DO NOT hold the Pen diagonally or horizontally while dispensing.
 - DO NOT use the Pen directly in the oral cavity
 - DO NOT dismantle the Pen.

■ PRECAUTIONS FOR MEDICAMENTS AND MATERIALS

- 1) Some materials and medicaments (hemostatic material) inhibit the adhesion of TOKUYAMA BOND FORCE II for an extended period even after careful meticulous cleansing with water. DO NOT USE products which contain:
 - eugenol,
 - diammine silver fluoride [molecular formula: $\text{Ag}(\text{NH}_3)_2\text{F}$],
 - phenols such as parachlorophenol, guaiacol, phenol,
 - aluminum chloride,
 - ferric sulfate,
 - aluminum sulfate,
 - epinephrine.
- 2) TOKUYAMA BOND FORCE II does not adhere to tooth structure immediately after applications of materials which contain:
 - hydrogen peroxide (oxydol),
 - sodium hypochlorite

We advise avoiding the use of these materials on the day of cementation, however these materials can be used for root canal treatment as the inhibition caused by these materials usually evaporates within five days.

■ STORAGE

- 1) Store TOKUYAMA BOND FORCE II at a temperature between 0 to 25 °C (32 to 77 °F). Storage under inappropriate condition will accelerate the degradation of TOKUYAMA BOND FORCE II.
- 2) If not in use for a prolonged period of time, store in a cool dark place.
- 3) Keep away from heat, direct sunlight, sparks and open flames.
- 4) DO NOT use after the indicated expiration date on the bottle/package.

■ DISPOSAL

Unused TOKUYAMA BOND FORCE II is to be absorbed into an inert absorbent material such as gauze or cotton, and disposed of in accordance with local regulations.

Refer to the packaging waste disposal guide shown on the end of this IFU.

■ CLINICAL PROCEDURES

1. Cleaning

Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.

2. Isolation

The rubber dam is the preferred method of isolation.

3. Cavity preparation

Prepare the cavity and rinse with water. Add bevels to the enamel margins of anterior preparations (class III, IV, V), as well as chamfers to the margins of posterior preparations (class I, II). Bevels and chamfers assist in erasing demarcations between the cavity margins and the restoration, enhancing both esthetics and retention.

- If there is an adherent surface including uncut enamel, apply etching agent to the uncut enamel.
- In case of porcelain/composite repairs, roughen the surface with a bur or a diamond point to prepare the area for adhesion; apply phosphoric acid for cleaning; rinse thoroughly with water; air dry thoroughly and treat with a silane coupling reagent according to its manufacturer's instructions.

4. Drying

Dry the cavity by blotting or with an air syringe.

- DO NOT desiccate the vital tooth. Desiccation can lead to post-operative sensitivity.
- The substances listed below inhibit curing and should be removed from the tooth surface by thoroughly cleaning the tooth surface with alcohol, citric acid, or the application of phosphoric acid for 2 to 3 seconds before application:
 - 1) Oil mist from handpiece,
 - 2) Saliva, blood and exudates.

5. Pulp Protection

Glass ionomer lining or calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT USE EUGENOL BASED MATERIALS to protect the pulp as these materials will inhibit the curing of TOKUYAMA BOND FORCE II.

6. Dispensing

6-1. Unit Dose

- Twist open the Unit Dose container and dip the disposable applicator.
- When opening the cap, make sure to verify the top and bottom of the container.
- To avoid cross infection, do not reuse the material on other patients.

6-2. Bottle

- Open the Bottle by removing the cap and dispense one or two drops of adhesive into the dispensing well. Immediately after dispensing, tightly replace the Bottle cap.
- Be sure to hold the Bottle of TOKUYAMA BOND FORCE II vertically when

dispensing. Do not hold the Bottle diagonally or horizontally while dispensing; this could cause the adhesive to flow back into the nozzle and become contaminated.


- Wipe off excess adhesive from the tip of the nozzle prior to closing.

6-3. Pen

Open the Pen by removing the cap when the cap is facing up and dispense one or two drops of adhesive into the dispensing well by holding the Pen vertically and pushing on button repeatedly until adhesive starts to flow (3 or 2 pushes, * This depends on how the button is pushed and the position that the Pen is being held). Immediately after dispensing, tightly replace the Pen cap.

- Be sure to hold the Pen vertically when dispensing. Do not hold the Pen diagonally or horizontally while dispensing; this could cause the adhesive to flow back into the nozzle and become contaminated.

- Wipe off excess adhesive from the tip of the nozzle prior to closing.

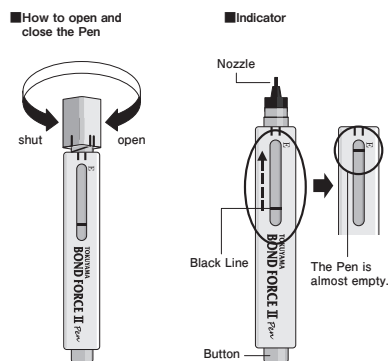Indicator:

The indicator allows users to monitor the remaining amount of adhesive. Black Line moves closer to the E as the amount of material decreases. When the line nears "E", this indicates that the Pen is almost empty. This is not an exact reading but only an indication of the remaining material.

7. Application

Using the disposable applicator, apply TOKUYAMA BOND FORCE II on the surface to be bonded and wait 10 seconds or more.

- Ensure TOKUYAMA BOND FORCE II covers all surfaces where the composite resin will be applied.
- Protect the dispensed adhesive and the inserted applicator from ambient light before the application.

Unit Dose: keep the applicator in the Unit Dose container.

Bottle, Pen: use a light blocking plate.

- Complete the application within 5 minutes after dispensing because TOKUYAMA BOND FORCE II contains a volatile alcohol.
- Be careful not to allow excess TOKUYAMA BOND FORCE II to flow into the subgingival. If TOKUYAMA BOND FORCE II flows into the subgingival, thoroughly rinse and dry the area.
- If the adherent surface include ceramics, porcelain or composite, prepare the surface according to the appropriate procedures prior to the application of TOKUYAMA BOND FORCE II

8. Air Dry

Using an oil-free air/water syringe, apply mild air continuously to the TOKUYAMA BOND FORCE II surface until the runny TOKUYAMA BOND FORCE II stays in the same position without any movement (usually for 5 seconds). Use a vacuum aspirator to prevent spatter of the adhesive.

- If accidental spattering occurs, it may cause the tissue to whiten or possible allergic reaction.
- If the adhesive pools on the cavity floor or the cavosurface angle and is too thick to air thin, blot the excess with a new disposable applicator before mild air application.
- If saliva, blood, or other fluids contaminate the applied adhesive, thoroughly rinse the cavity with water, dry and re-apply new adhesive.
- Do not rinse the applied adhesive with water except in unintentional contamination.

9. Light-cure

Light-cure the surface for 10 seconds or more, keeping the curing light tip within a distance of 2 mm from the surface. If the cavity is too large or the light tip is too distant (e.g. MOD), divide the area into segments and light-cure each segment individually.

- Confirm that the light-curing unit has sufficient intensity ($>300\text{mW}/\text{cm}^2$) before using. Note that using a cracked light guide will lower the intensity.

10. Light-cured Composite

Restore with light-cured composite resin according to its manufacturer's instructions. The composite resin must be finished and polished.

- When dual-cured composite resins are placed into a cavity, the first increment must be light-cured using a layering technique.
- Do not use self-cured composite resins here because the Phosphoric acid monomer contained in TOKUYAMA BOND FORCE II may interfere with the curing of the self-cured resins, leading to their premature detachment.

The user and/or patient should report any serious incident that has occurred in

relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use. Specifications are subject to change without notice. When the product specifications change, the instructions and precautions may also change.

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Resin-based Dental Restorative Material

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Packaging waste disposal guide

(According to European Commission Decision 97/129/EC)

ENGLISH

Packaging waste disposal guide

SYRINGE or PLT (cap and body)	Dispose of as industrial waste.
Bottle (cap and body)	Dispose of as industrial waste.
Packaging bag for shadeguide and/or PLT	LDPE4 (Low density polyethylene):Recycling. Follow local instructions for disposal.
Case body for applicators	CP7 (cellulose propionate):Recycling. Follow local instructions for disposal.
Case cap for applicators	PP5 (Polypropylene): Recycling. Follow local instructions for disposal.
Paperbox	PAP21 (Non-corrugated fibreboard):Recycling. Follow local instructions for disposal.
Partation	PP5 (Polypropylene): Recycling. Follow local instructions for disposal.
IFU and guide	PAP22 (Paper): Recycling. Follow local instructions for disposal.

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