Adhesive Resin Cement

ESTEC EM II PLUS

CE 1639

Instructions for Use

**Tokuyama Dental Corporation**

38-9, Taitou 1-chome, Taitou-ku, Tokyo 110-0016, Japan

Tel: +81-3-3835-7201

URL: <https://tokuyama-dental.com>

ENGLISH	2	БЪЛГАРСКИ	8	ČESKY	16	DEUTSCH	23
DANSK	32	EESTI KEEL	39	ESPAÑOL	46	SUOMI	53
FRANÇAIS	60	ΕΛΛΗΝΙΚΑ	68	HRVATSKI	76	MAGYAR	83
ITALIANO	90	LIETUVIŲ K.	98	LATVISKI	105	NEDERLANDS	112
NORSK	119	POLSKI	126	PORTUGUÊS	134	ROMÂNĂ	142
SVENSKA	149	SLOVENČINA	156	SLOVENŠČINA	163	TÜRKÇE	170

This product is certified as a medical device under the legislation of the European Union on medical devices by SGS CE1639, exclusively for the indication(s) shown in this IFU. Other non-medical uses ascribed to this product are not within the scope of CE certification, and users should be aware product performance and/or safety has not been evaluated by SGS for those purposes.

Read all information, precautions and notes before using.

PRODUCT DESCRIPTION AND GENERAL INFORMATION

1. ESTECCEM II PLUS is a dual-cure (light and/or self-cure), radiopaque, adhesive resin cement system with excellent handling, esthetic and adhesive properties to teeth and all prosthetic materials. It has radiopacity equal to or greater than 1mm aluminum, and is classified as a Type 2 and Class 3 material by ISO 4049. The main components of the ESTECCEM II PLUS KIT are the PASTE (A/B) and Tokuyama Universal Bond II (A/B). The PASTE is dispensed in an automix syringe and is available in various shades (UNIVERSAL, CLEAR, BROWN, WHITE OPAQUE). Tokuyama Universal Bond II promotes the adhesion of PASTE to tooth structure. Tokuyama Universal Bond II also enhances the adhesion of PASTE to all prosthetic materials.
2. The PASTE contains Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate (TEGDMA), Bisphenol A polyethoxy methacrylate (Bis-MPEPP), Peroxide, Camphorquinone, N,N-dimethyl-p-toluidine and Silica-Zirconia Filler (Filler Load 74 wt%).
3. The PASTE A contains N,N-dimethyl-p-toluidine as polymerization initiator in a concentration above 0.1% weight by weight. N,N-dimethyl-p-toluidine is classified as carcinogenicity 1B, in accordance with Part 3 of Annex VI to Regulation (EC) No 1272/2008 of the European Parliament and of the Council. Tokuyama Dental reviewed in detail the documents prepared by the regulatory authorities (Risk Assessment Committee of European Chemical Agency) and also conducted a risk assessment regarding ESTECCEM II PLUS. It is concluded that ESTECCEM II PLUS does not cause an increased risk of carcinogenicity, as the estimated exposures of patients treated with ESTECCEM II PLUS in dental procedures to N,N-dimethyl-p-toluidine are not expected to influence the cancer incidence in humans.
4. Tokuyama Universal Bond II contains Phosphoric acid monomer, Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate (TEGDMA), 2-Hydroxyethyl methacrylate (HEMA), MTU-6 (thiouracil monomer), Silane coupling agent, Peroxide, Borate catalyst, Acetone, Ethanol and Purified water.

INDICATIONS

- Cementation of crowns, bridges, inlays, and onlays made of glass/oxide ceramics (porcelain, zirconia and alumina), metals/alloys (precious and non-precious) and resin materials including inorganic filler (composite materials).
- Repair of fractured porcelain fused to metal crowns and all ceramic restorations.
- Cementation of veneers.
- Cementation of adhesion bridges.
- Cementation of metal or resin cores, metal or glass-fiber posts.

CONTRAINDICATIONS

ESTECCEM II PLUS contains methacrylic monomers, peroxide, organic solvents and acids. DO NOT use ESTECCEM II PLUS for patients allergic to or hypersensitive to methacrylic monomers, related monomers, peroxide, organic solvents and acids.

PRECAUTIONS

- 1) Use Tokuyama Universal Bond II as primer for the ESTECCEM II PLUS PASTE. DO NOT use other primers or adhesives as primer for the adhesive surfaces.
- 2) DO NOT use ESTECCEM II PLUS for any purpose other than those listed in these instructions. Use ESTECCEM II PLUS only as directed herein.
- 3) ESTECCEM II PLUS is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.

- 4) DO NOT use the ESTECCEM II PLUS KIT, PASTE, or Tokuyama Universal Bond II if the safety seals are broken or appear to have been tampered with.
- 5) If ESTECCEM II PLUS causes an allergic reaction or oversensitivity, discontinue its use immediately.
- 6) Use examination gloves (plastic, vinyl or latex) at all times when handling ESTECCEM II PLUS to avoid the possibility of allergic reactions from methacrylic monomers.
Note: Certain substances/materials may penetrate through examination gloves. If the PASTE, or Tokuyama Universal Bond II comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- 7) Avoid contact of the PASTE, or Tokuyama Universal Bond II with eyes, mucosal membrane, skin and clothing.
 - If the PASTE, or Tokuyama Universal Bond II comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If the PASTE, or Tokuyama Universal Bond II comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration is seated. Tokuyama Universal Bond II affected areas may whiten from protein coagulation, but such whitening should disappear within 24 hours. If such whitening does not disappear within 24 hours, immediately contact a physician, and the patient should be so advised.
 - If the PASTE or Tokuyama Universal Bond II comes into contact with the skin or clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze.
 - Instruct the patient to rinse his mouth immediately after treatment.
- 8) The PASTE or Tokuyama Universal Bond II should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- 9) To avoid the unintentional ingestion of the PASTE or Tokuyama Universal Bond II, do not leave it unsupervised within the reach of patients and children.
- 10) DO NOT expose Tokuyama Universal Bond II or the vapor to open flame because Tokuyama Universal Bond II is flammable.
- 11) DO NOT use mixing wells supplied by other manufacturers.
- 12) To avoid cross infection or lowering bond strength, DO NOT reuse the disposable applicator and the disposable mixing well. Clean the mixing well thoroughly with alcohol after each use.
- 13) DO NOT use ESTECCEM II PLUS directly to the cavity being in close proximity to the pulp. Pulp protection with glass ionomer lining or calcium hydroxide is recommended. DO NOT use EUGENOL BASED MATERIALS to protect the pulp as these materials will inhibit the curing of Tokuyama Universal Bond II.
- 14) When using a light-curing unit, protective eye shields, glasses or goggles should be worn at all times.
- 15) Be aware that when a restoration is chipped from misaligned occlusion or bruxism (clenching, grinding or tapping), the repaired restoration may chip again.
- 16) DO NOT mix the PASTE or Tokuyama Universal Bond II with other brands of paste, primers or adhesives.
- 17) The PASTE or Tokuyama Universal Bond II should be brought to room temperature before using. After removing from the refrigerator, allow the product to sit for 20 minutes or until it reaches room temperature. Return the PASTE to the refrigerator after using.

■ PRECAUTIONS FOR PASTE

- 1) Use the Cement Mixing Tip or Mini Mixing Tip S included in this KIT when extruding the mixed PASTE.
- 2) Use a clean instrument or explorer for the application or removal of the PASTE.
- 3) To avoid unexpected hardening of the PASTE, after each use remove the mixing-tip from the automix syringe. Using a clean cloth or tissue wipe the excess paste from the tip of syringe

prior to recapping the syringe using the cap provided with the syringe at the time of purchase.

■ PRECAUTIONS FOR Tokuyama Universal Bond II

- 1) To avoid splashing the liquid, the bottles of Tokuyama Universal Bond II should not be opened immediately after taking out of the refrigerator.
- 2) Use the mixing well or disposable mixing well provided when dispensing and mixing of Tokuyama Universal Bond II. If Tokuyama Universal Bond II is not dispensed into the rubber mixing it may be difficult to remove the cured Tokuyama Universal Bond II from mixing well. The cured Tokuyama Universal Bond II can be removed by inverting the rubber.
- 3) Some materials and medicaments (hemostatic material) inhibit setting/adhesion of Tokuyama Universal Bond II. DO NOT use products that contain:
 - eugenol,
 - aluminum chloride,
 - ferric sulfate,
 - calcium sulfate,
 - aluminum sulfate,
 - diammine silver fluoride [molecular formula: $\text{Ag}(\text{NH}_3)_2\text{F}$]
- 4) Tokuyama Universal Bond II does not adhere to tooth structure immediately after applications of materials that contain:
 - hydrogen peroxide (oxydol),
 - sodium hypochlorite

We advise avoiding the use of these materials on the day of cementation, however these materials can be used for root canal treatment as the inhibition caused by these materials usually evaporates within five days.

- 5) Be sure to hold the bottles of Tokuyama Universal Bond II A and B vertically when dispensing. Do not hold the bottle diagonally or horizontally while dispensing; this could cause the Tokuyama Universal Bond II to flow back into the nozzle and become contaminated.

■ STORAGE

- 1) Store the ESTEC EM II PLUS KIT, PASTE under refrigeration at temperatures between 0 to 10°C (32 to 50°F).
- 2) Store the Tokuyama Universal Bond II at temperatures between 0 to 25°C (32 to 77°F).
- 3) Keep away from heat, direct sunlight, sparks and open flames.
- 4) DO NOT use after the indicated expiration date on the syringe/bottle/package.

■ DISPOSAL

- To dispose of the PASTE, extrude unused paste from the automix syringe, mix and allow paste to harden before disposal.
- Unused Tokuyama Universal Bond II is to be absorbed into an inert absorbent material such as gauze or cotton, and disposed of in accordance with local regulations.
- Do not allow to enter drains, sewers or watercourses.
- Must not be disposed of together with household garbage.
- Follow local instructions for disposal.
- Refer to the packaging waste disposal guide shown on the end of this IFU.

■ CLINICAL PROCEDURES

1. Tooth Preparation

Remove contamination from the tooth surface such as plaque, temporary cement residue, oil from materials to test crown fit, oil mist from a hand-piece, saliva, blood and exudates fluids by brushing with flour pumice, ultrasonic scaling or cleaning the surface of the tooth with alcohol. Thoroughly rinse and dry the surface.

- If the adherent surface involves uncut enamel, apply etching agent to the uncut enamel. See

section “Selective-enamel-etch technique”.

- DO NOT desiccate the vital tooth. Desiccation can lead to post-operative sensitivity.

2. Etching of the tooth structure (optional)

(a.) Selective-enamel-etch technique

Apply phosphoric acid gel only to the enamel surrounding the margin of prepared cavity and leave phosphoric acid gel in place for 10-15 seconds. Rinse the etched surface thoroughly (at least 15 seconds) with water, and then dry with mild air.

(b.) Total etch technique

Apply phosphoric acid gel onto the prepared enamel first, and then the dentin. The etchant should be left to react on the enamel and dentin for 10-15 seconds. Rinse thoroughly (at least 15 seconds) with water and dry with mild air or with cotton pellets; do not overdry.

3. Restoration Preparation

Follow the procedure shown below, depending on the material to be restored.

1) Ceramics other than Porcelain, and Composite materials

Roughen the interior of the restoration by sandblasting (0.1 to 0.2MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion. Thoroughly rinse and dry the surface.

2) Porcelain

DO NOT roughen the surface; Prepare the interior of the restoration, providing a clean and fresh surface for adhesion. Thoroughly rinse and dry the surface.

3) Metal restorations

Roughen the area by sandblasting (0.3 to 0.5MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion. Thoroughly rinse and dry the surface.

- When using the sandblaster or air abrasion, use at least 30 to 50 micron grid of alumina powder.

- If unable to roughen or prepare the adhesive surface, etch the area for adhesion with phosphoric acid then thoroughly rinse and dry.

- The substances listed below inhibit adhesion and must be removed from the adhesive surface by mechanical cleaning or alcohol soaking. Clean the restoration after adjustment is complete.

- 1) Gypsum and investment

- 2) Silicone oil from fit check material

- 3) Oil mist from handpiece

- 4) Plaque, tartar, saliva, blood and exudate fluids

- 5) Temporary restoration and temporary cement

- If necessary, protect the surface not being bonded with metal separator, paraffin film, Vaseline, etc.

4. Dispensing of Tokuyama Universal Bond II

Allow Tokuyama Universal Bond II to reach room temperature prior to use. Open the bottle by removing the cap and dispense one drop each of Tokuyama Universal Bond II A and B into mixing well or disposable mixing well. Mix thoroughly with a disposable applicator until the mixed bonding agent turns green.

- Dispense Tokuyama Universal Bond II B IMMEDIATELY after dispensing A. Tokuyama Universal Bond II A may turn pink due to solvent volatilization if allowed to sit too long. If Bond A turns pink, dispose of it and re-dispense fresh Tokuyama Universal Bond II A and B.

- Complete the application within 1 minute of dispensing when using the mixing well since Tokuyama Universal Bond II contains volatile solvents. After mixing, the color of Tokuyama Universal Bond II changes gradually based on the reaction of the catalyst of Tokuyama Universal Bond II, this is normal and does not indicate a problem with Tokuyama Universal

Bond II.

- Complete the application within 3 minutes of dispensing when using the disposable mixing well.
- DO NOT touch the nozzle to the mixing well while dispensing. DO NOT interchange the bottle caps of Tokuyama Universal Bond II A and B. If Tokuyama Universal Bond II A and B come into contact while dispensing, Tokuyama Universal Bond II may cure in the nozzle.
- Wipe off excess Tokuyama Universal Bond II from the tip of the nozzle and the inside of the cap prior to closing.
- Immediately after dispensing, tightly replace the bottle caps of Tokuyama Universal Bond II A and B.
- Do not over tighten the bottle caps.

5. Restoration Pretreatment with the Tokuyama Universal Bond II

1) Application

Using the disposable applicator, apply the mixed Tokuyama Universal Bond II on the surface to be bonded.

- Ensure Tokuyama Universal Bond II covers all surfaces where the PASTE will be applied.
- DO NOT over apply. One application of Tokuyama Universal Bond II is sufficient. Too thick of a layer may compromise the fit of the restoration.
- DO NOT allow saliva, blood, oil, water etc. to come in contact with the surface to be bonded. If the treated surface is contaminated, thoroughly rinse with water, dry and re-apply fresh Tokuyama Universal Bond II.
- DO NOT wait too long after the application of Tokuyama Universal Bond II. Air dry within 30 seconds after application, to ensure the appropriate film thickness.

2) Air dry

Using an oil-free air/water syringe, apply mild air to the surface. Use a vacuum aspirator to prevent spatter of the Tokuyama Universal Bond II.

- Be sure to dry the surface after Tokuyama Universal Bond II is applied. Incomplete drying may result in poor adhesion.

6. Tooth (Adherent surface) Pretreatment with the Tokuyama Universal Bond II

1) Application

Using the disposable applicator, apply the mixed Tokuyama Universal Bond II on the surface to be bonded.

- Ensure Tokuyama Universal Bond II covers all surfaces where the PASTE will be applied. Even if the adherent surface includes materials other than tooth structure such as ceramics, porcelain, composite, and/or metal, no extra primers are needed.
- Dispensed Tokuyama Universal Bond II can be used for both the restoration and tooth pretreatment if used within 1 minute using the mixing well or 3 minutes if using the disposable mixing well.
- If the time after dispensing is over 1 minute (mixing well), or over 3 minutes (disposable mixing well), DO NOT use dispensed Tokuyama Universal Bond II due to the possibility of the poor coating properties or insufficient adhesion. If the time after dispensing exceeds the limit mentioned above (1 minute or 3 minutes), dispense again one drop each of Tokuyama Universal Bond II A and B into the mixing well or disposable mixing well and mix thoroughly with a disposable applicator.
- If the Tokuyama Universal Bond II pools on the cavity floor or a cavosurface angle and is too thick to air thin, remove the excess with a paper point or a new disposable applicator.
- Be careful not to allow excess Tokuyama Universal Bond II to flow into the subgingival. If Tokuyama Universal Bond II flows into the subgingival, thoroughly rinse and dry the area.
- DO NOT wait too long after the application of Tokuyama Universal Bond II; this will lead to the possibility of the restoration not seating properly with cementation. Air dry within 30

seconds after application, to ensure the appropriate film thickness.

2) Air dry

Using an oil-free air/water syringe, apply weak air continuously to the Tokuyama Universal Bond II surface until the runny Tokuyama Universal Bond II stays in the same position without any movement. Finish by using mild air to the surface. Use a vacuum aspirator to prevent spatter of the Tokuyama Universal Bond II.

- If saliva, blood, or other fluids contaminate the applied Tokuyama Universal Bond II, thoroughly rinse with water, dry and re-apply fresh Tokuyama Universal Bond II.
- DO NOT rinse the applied Tokuyama Universal Bond II with water except in unintentional contamination.
- If accidental spattering occurs, it may cause the tissue to whiten or possible allergic reaction.

7. Cementation and Final Bonding

1) Preparing the syringe and mixing tips

Before each use, remove the cap from automix syringe, and squeeze out a small quantity of paste A and B, making sure equal amounts are being dispensed through the two outlets of the syringe.

- How to remove the cap: turn the cap 1/4 of a turn counterclockwise to align the projections on the cap with the grooves in the syringe. Holding the base of the cap, remove the cap by twisting and pressing downward.
- DO NOT allow PASTE A to come into contact with PASTE B, there is a possibility of hardening of the PASTE at the tip of syringe. Wipe the excess PASTE from the tip of syringe with a clean cloth or tissue before using a new mixing tip.
- If equal amounts of PASTE A and B are not dispensed, there is a possibility of poor polymerization. Ensure equal amounts are being dispensed through the two outlets of the syringe prior to attaching a mixing tip.
- How to attach the mixing tip: align the projections on the adapter with the grooves in the syringe as closely as possible, and turn the mixing tip 1/4 of a turn clockwise to lock it in place.
- After use, remove the mixing tip from automix syringe and replace the cap of the automix syringe.

2a) Cementation of Restoration

Attach a Cement Mixing Tip to the syringe, squeeze PASTE (A and B) from the syringe and apply the mixed PASTE to the inner surface of the restoration. Place the restoration on the tooth with firm pressure. Any excess PASTE at the margin can be removed with a brush tip.

- DO NOT apply the PASTE directly to the prepared tooth or the cavity treated with Tokuyama Universal Bond II to avoid incorrect seating of the restoration.
- To keep the appropriate film thickness, seat the restoration as soon as possible after the application of the PASTE (within 1 minute after dispensing), if not there is a possibility that the restoration will not seat properly with cementation.

2b) Placing the core or post

Attach a Mini Mixing Tip S and a Nozzle for Mini Mixing Tip to the syringe for application in the root canal. Apply the mixed paste over the entire adherent surface of the core or post, or the entire tooth surface within the cavity. Place the core or post quickly into the cavity, slightly vibrating it to prevent air bubbles from entering the root canals.

- If the paste is applied directly into the cavity, place the core or post into the cavity within 30 seconds after application of the PASTE.
- DO NOT use Lentulo spirals to insert the cement in the root canal as this can excessively accelerate setting.

8. Removal of Excess PASTE

There are two methods for removal of excess PASTE.

a) When using the light-cure method

Once the restoration is seated, light-cure the excess PASTE with a dental curing light for 2 to 4 seconds. The gel state of the excess PASTE can be easily removed.

- Excess light curing will lead to difficult removal of the excess cement.
- The Tokuyama Universal Bond II layer may appear fluorescent under the dental curing light irradiation.

b) When using the self-cure method

The gel state of the excess cement can be easily removed within 1 to 3.5 minutes after seating.

- Waiting more than 3.5 minutes after seating will lead to difficult removal of the excess cement.
- The setting time of the PASTE may change (approximately 20 seconds at the maximum) depending on the oral temperature in each clinical case.

9. Final Adjustment

1) Final PASTE hardening

a) In case of translucent restoration materials such as ceramics or composite materials.

Irradiate the light to the applied PASTE on the seated restoration for 20 seconds or more. When the adherent area is too large to be completed by a single light-cure, multiple cures are recommended.

- Confirm that the light-curing unit has sufficient intensity ($>500\text{mW/cm}^2$) before using.

Note that using a cracked light guide will lower the intensity.

b) In case of non translucent restoration materials such as metal.

Light-cure along margins for 20 seconds or more, then allow the PASTE to set for 8 minutes with the patient applying normal occlusal pressure to the restoration.

2) Finishing

Check patient's bite, if necessary. Re-polish the restoration with material of choice.

The user and/or patient should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use.

Specifications are subject to change without notice. When the product specifications change, the instructions and precautions may also change.

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Packaging waste disposal guide

(According to European Commission Decision 97/129/EC)

ENGLISH

Packaging waste disposal guide

Bottle (cap and body)	Dispose of as industrial waste.
SYRINGE (cap and body)	Dispose of as industrial waste.
Case body for applicators	CP7 (Cellulose propionate): Recycling. Follow local instructions for disposal.
Case cap for applicators	PP5 (Polypropylene): Recycling. Follow local instructions for disposal.
Packaging bag for tip	LDPE4 (Low density polyethylene): Recycling. Follow local instructions for disposal.
Paperbox and/or partation	PAP21 (Non-corrugated fibreboard): Recycling. Follow local instructions for disposal.
IFU and/or guide	PAP22 (Paper): Recycling. Follow local instructions for disposal.

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