## TOKUPOST.18S

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## INSTRUCTIONS FOR USE OF ENDOCANALAR TRANSLUCENT QUARTZ FIBER POSTS

**PRODUCT / TYPE** Quartz fiber posts  
**INTENDED USE** Endodontic posts for reconstruction  
**MEDICAL DEVICE**

Class IIa Medical Device - Long-term implantable. Compliant with EU Regulation 2017/745.

### 1. DESCRIPTION

TOKUPOST Iperquartz is a highly translucent quartz fiber radio-opaque post, with variable conicity. It is available in two lengths and shapes: 18S with 18 mm length, 15R with 15 mm length.

**2. INDICATIONS FOR USE**

- The use of these devices is intended only for odontologists and odontostomatologists;
- In case of insufficient residual dental substance (< 4 mm), TOKUPOST Iperquartz is suitable to sustain the restoration of coronal walls.

### 3. CONTRAINDICATION

In case of insufficient residual dentine, ("ferrule" effect), less than 2 mm, the use of posts is not recommended.

### 4. WARNINGS AND STORAGE

TOKUPOST Iperquartz posts are disposable medical devices; it is absolutely forbidden to reuse an already used post. This would entail not only serious risks of bacterial contamination and infections transmission, but also of the collapse of the post already subject to previous physical-mechanical stress. Store posts protected from light and heat sources.

### 5. PRECAUTIONS

It is recommended to clean the post with alcohol before placing it into the canal, in order to "decrease" it if it has been decontaminated. The TOKUPOST Iperquartz post is silanized. If necessary shorten the post outside the oral cavity (TOKUPOST 18S).

### 6. UNWANTED REACTIONS

Based on literature available to date, no unwanted reactions are known.

### 7. DISPOSAL OF THE DEVICE

The device must be disposed of according to local regulations, considering that it is special hospital waste.

### 8. SAFETY AND LIABILITY

Micro.Medica srl and Tokuyama Dental accept no liability or claim for damages due to:  
 • use of instruments not belonging to the system and which might compromise its operation;  
 • non compliance with the instructions for use.

**! The user is liable for using materials for uses different from the ones specified in the instructions for use.**

### 9. STEP BY STEP USE PROTOCOL: STEPS FROM ROOT CANAL PREPARATION TO CORE BUILD-UP RECONSTRUCTION

1. Prepare the root canal as per the known technique; make sure that there is enough space to place the endodontic post;
2. Tokuyama Dental recommends to use a rubber dam before treating the tooth;
3. Carry out the initial removal of gutta-percha with the usual instruments, complete the canal widening with a reamer;
4. Select the TOKUPOST post based on clinical evaluations and with the help of the X ray to select the type and the most appropriate size among the available shapes, depending on the tooth anatomy;
5. Determine the length of the coronal build-up, which must be about 1/3 of the final length of the post. Remove the root canal filling leaving the gutta-percha seal untouched on the apical third (3-5 mm) obtaining in this way a space to house 2/3 of the post length; in case of curved canals this length must be reconsidered;
6. Complete the preparation of the post's housing with the TOKU DRILL calibrated bur depending on the size of the selected post (low revolution speed, never exceeding 15,000 rpm); Use TOKUPOST VERIFIER to confirm that the post has seated correctly in the canal and define its length;
7. If necessary cut the post outside the oral cavity with a diamond separating disc at low speed. Don't use forceps or nippers because the pressure action would "strip" the post damaging its structure;
8. Clean the post with alcohol to remove impurities;
9. Complete the preparation of the post's housing with the TOKU DRILL calibrated bur depending on the size of the selected post (low revolution speed, never exceeding 15,000 rpm); Use TOKUPOST VERIFIER to confirm that the post has seated correctly in the canal and define its length;
10. Clean the canal in the most appropriate way, etch the dentine and the entrance opening following the etchant manufacturer's instructions;
11. Rinse the canal with plenty of water making sure to remove all the etchant (it is recommended to use an irrigation needle);
12. Dry and blot the canal with paper cones and prepare for the following cementing steps (e.g. ESTECEM II Plus Kit);
13. In the special dappen mix in a 1:1 ratio the two components UNIVERSALBOND II A and UNIVERSAL BOND II B, with the help of a brush apply the primer on the post's surface and gently dry for 5 seconds;
14. With the same mix of UNIVERSAL BOND II, apply and wet with the adhesive all the surfaces inside the root canal with a brush
15. Following the relevant instructions for use, mix the cement ESTECEM II Plus with the special tips (supplied with the ESTECEM II Plus kit). Depending on the preferred technique, apply a cement layer on the post's surface or with a special mixing tip extrude the cement in the canal. Proceed to positioning the post into the canal
16. Place the post into the canal and gently push it all the way down. Shape any excess cement around the post, light cure for 20 seconds or more keeping the post in place until the cement has completely hardened;
17. Shape ESTELITE UNIVERSAL Flow Super Low for the build-up reconstruction following the instructions for use.

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